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EN INSTRUCTION MANUAL

Dries and cares your hearing systems

Moisture and ear wax can impair the function of hearing systems and cause failures. The **dry-star uv** offers optimal care. Comfort drying at 55°C extends life time significantly. Ultraviolet light (253,7nm UV-C) eliminates 99,9% of germs and bacteria. This clinical cleanliness prevents ear infections risks and protects both your hearing systems as well as your health.

Using first time

- Insert the round plug of the supplied 12V power adapter into the jack on the rear side of your drying station
- Then plug the power supply into an AC power socket

Operation

- Open the lid of your **dry-star uv**
- Place the with wet wipes pre-cleaned hearing systems with the battery-door open (if possible) into your drying station
- Briefly touch the sensor button and the comfort drying program (drying + UV-C) is starting automatically
 - The 3 hours drying program with approx. 55°C warm air is fully processor controlled
- Close the lid
 - Ultraviolet light (253,7 nm) eliminates germs and bacteria to 99.9% within 8 minutes
 - After 8 min the UV-C light switch off automatically, the drying process continues
 - Within the last 8 minutes of the 3 hours drying program, a second UV-C hygiene runs
 - After 3 hours the drying station automatically shutdown to stand-by
- Open the lid again
- Remove the dried and hygienically cleaned hearing systems

Maintenance and operation

- We recommend wet wipes for cleaning your **dry-star uv**
- You can interrupt the drying process by holding the sensor button for more than 3 seconds
- The **dry-star uv** is very energy efficient; unplug the power supply only for prolonged absence

Warnings

- Avoid contact with water
- Don't use the **dry-star uv** near basins etc.
- In the event of a malfunction, unplug the power supply and take the drying station to your retailer
- Don't disassemble the drying station and use only the supplied 12V power adapter
- Children must be supervised to ensure that they don't use the drying station as a toy
- The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station
- CAUTION: Do not look into the UV-C light, as this can harm your eyes and your skin
- The drying station has an automatic room and temperature modulation as well as safety shutdowns, when the lid is opened during UV-C program

| | |
|----|-------------------------|
| DE | BEDIENUNGSANLEITUNG |
| EN | INSTRUCTION MANUAL |
| FR | MANUEL D'UTILISATION |
| ES | MANUAL DE INSTRUCCIONES |
| PT | MANUAL DE INSTRUÇÃO |
| IT | MANUALE D'USO |
| NL | HANDLEIDING |
| TR | MANUALE D'ISTRUZIONI |
| DK | BETJENINGSVEJLEDNING |
| FI | KÄYTTÖOHJEET |
| RU | ИНСТРУКЦИЯ |

dry★uv

dry-star uv

Electronic drying station for hearing aids
Perfect drying and care
Hygienic cleansing by UV-C light

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