TROCKENETUI

DRY CASE

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DROOGETUI

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ESTUCHE SECADOR

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User instructions

Mode d'emploi

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DEUTSCH

Cedis Trockenetui

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ENGLISH

Cedis dry case

Drying and protection all in one. In the *Cedis* dry case your hearing system is in good hands. It maintains the function of your hearing system in the long term.

Moisture in the hearing aid, e.g. caused by sweat, leads to an early wear out of the high quality electronics. In the *Cedis* dry case hearing aids and earmolds will be dried gently and safely. The operating life of your hearing aid increases substantially.

How it works:

Please remove the battery from the hearing aid before starting the drying process. Put the hearing aid and earmold in the case. For an effective drying process leave the battery compartment open. Connect the power supply with the source of power. The dry case is ready. It takes only 3 hours until your hearing aid has been dried gently and safely. A light-emitting diode (red) serves as a monitor for the drying process. The dry case can remain switched on during the night, as current consumption is very low. Disconnect the case from the power supply after the drying process in order to protect it from damage.

Cleaning of the case:

Simply clean with a damp cloth inside and outside.

Technical description:

Only use the 12 V power supply unit included or optional with the available 12 V adapter for cigarette lighter (REF 87443) for using it on the way, in the camper or car. The dry case is made of shock resistant plastic. Capacity: 1 W/12 V. Average drying temperature 40 °C (104 °F), maximum temperature up to 50 °C (122 °F) is possible.

Guarantee:

1 year after purchase date when using it appropriately. Please enclose the sales slip.

Caution:

Only use for drying of hearing aids and earmolds. The unit radiates heat on the bottom part. Do not use on surfaces, that react heat sensitive. Do not cover the case or use it in water. When showing signs of damage or after improper use return the dry case to the sales point for control! This unit is only designed for domestic use and not for medical purposes.

Cedis dry case

The professional product from the proven *Cedis* hygiene line.

Cedis® ... the brand-care for hearing systems

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egger Otoplastik + Labortechnik GmbH