DRY-SPACE UV

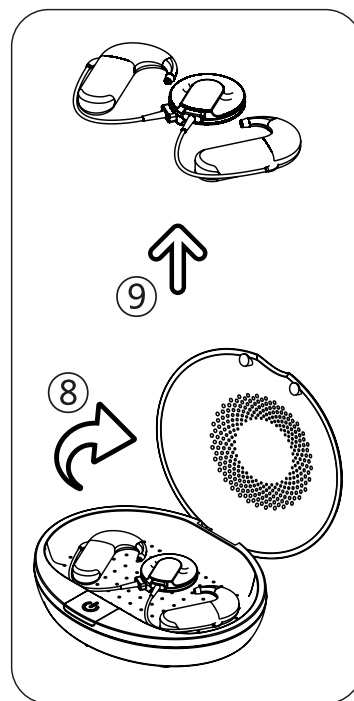
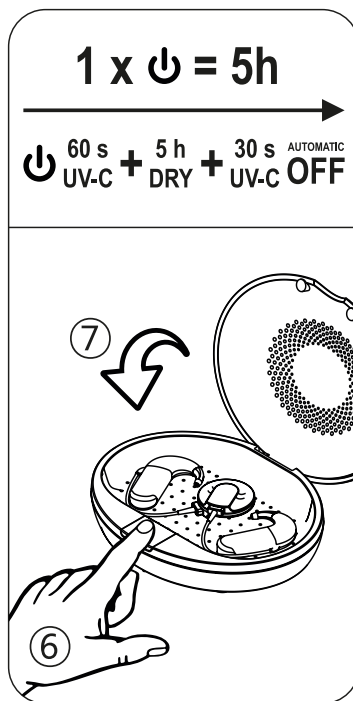
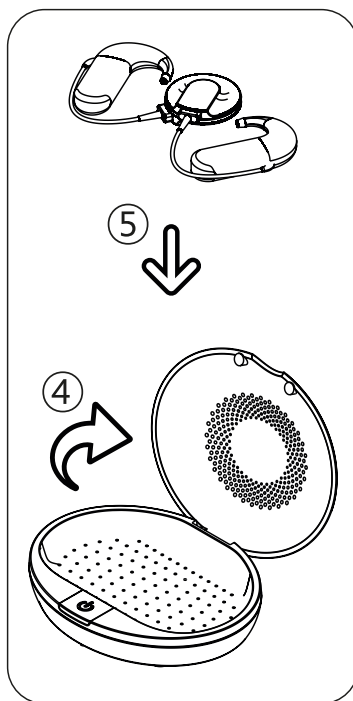
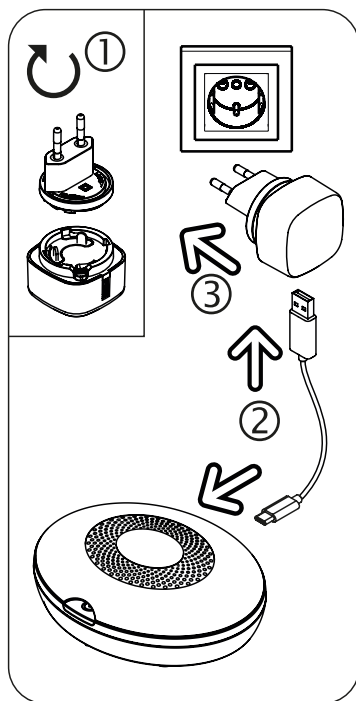
Electronic drying station for cochlear implants.

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EN INSTRUCTION FOR USE

Dries and cares your hearing systems and implants

Moisture and ear wax can impair the function of hearing systems and cause failures. The DRY-SPACE UV offers optimal care. Comfort drying at 113°F (45°C) extends life time significantly. Ultraviolet light (253,7nm UV-C) eliminates 99,9% of germs and bacteria. This clinical cleanliness prevents ear infections risks and protects both your hearing systems as well as your health.

Using first time

1. Insert your country-specific adapter into the USB power supply by rotating it clockwise into the slot.
2. Plug the USB cable into the USB power supply unit (5V/2.5A) and the USB-C plug into the USB-C port on the back of your DRY-SPACE UV.
3. Then plug the USB power supply unit into a mains socket.

① The DRY-SPACE UV is ready to use

Operation

4. Open the lid of your DRY-SPACE UV
5. Remove the batteries from your battery pack, keep the battery pack cover separate and place the hearing systems into your DRY-SPACE UV
6. Briefly press the button and the comfort drying program (drying + UV-C) is starting automatically
 - The 5 hours drying program with approx. 113°F (45°C) warm air is fully processor controlled
7. Close the lid
 - Ultraviolet light (253,7 nm) eliminates germs and bacteria to 99.9% within 1 minutes
 - After 1 min the UV-C light switch off automatically, the drying process continues
 - Within the last minute of the 5 hours drying program, a second UV-C hygiene runs
 - After 5 hours the drying station shutdown to stand-by automatically
8. Open the lid again
9. Remove the dried and hygienically cleaned hearing systems

① LED permanent light

① LED blink light

① LED permanent light

① LED blink light

① LED switch off

Maintenance and operation

- We recommend wet wipes for cleaning your DRY-SPACE UV
- You can interrupt the drying process by holding the button for more than 3 seconds
- The DRY-SPACE UV is very energy efficient; unplug the power supply only for prolonged absence

Warnings

• Avoid contact with water • Don't use the DRY-SPACE UV near basins etc. • In the event of a malfunction, unplug the power supply and take the drying station to your retailer • Don't disassemble the drying station and use only the supplied 5 V power adapter • Children must be supervised to ensure that they don't use the drying station as a toy • The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station • CAUTION: Do not look into the UV-C light, as this can harm your eyes and your skin • The drying station has an automatic room and temperature modulation as well as safety shutdowns, when the lid is opened during UV-C program

- Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities

- Contact your local government for information regarding the collection systems available

- If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being

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© 2010-2025 flow-med GmbH. All rights reserved.
flow-med GmbH
Otoplastic AG