



dry-cap uv 4[®]

Smart electronic drying cap for all hearing aids.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	2 - 5
DE	GEBRAUCHSANWEISUNG	6 - 9
FR	MODE D'EMPLOI	10 - 13
ES	MANUAL DE INSTRUCCIONES	14 - 17
PT	MANUAL DE INSTRUÇÕES	18 - 21
IT	MANUALE DI ISTRUZIONI	22 - 25
NL	GEBRUIKSAANWIJZING	26 - 29
GR	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ	30 - 33
TR	KULLANIM KILAVUZU	34 - 37
DK	BRUGERVEJLEDNING	38 - 41
SE	BRUKSANVISNING	42 - 45
NO	BRUKSANVISNING	46 - 49
PL	INSTRUKCJA OBSŁUGI	50 - 53
KR	사용 지침	54 - 57
CN	用法說明	58 - 61
JP	取扱説明書	62 - 65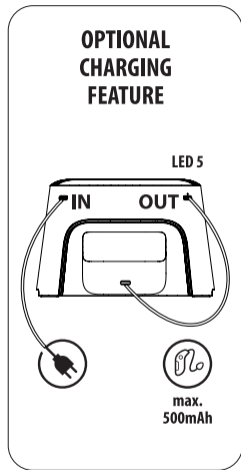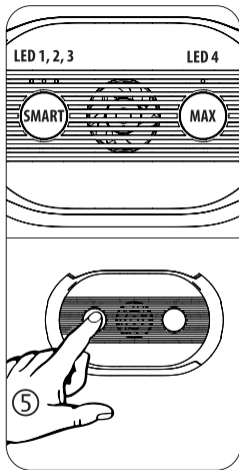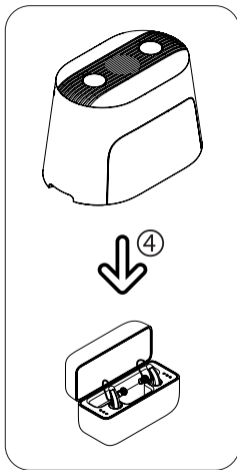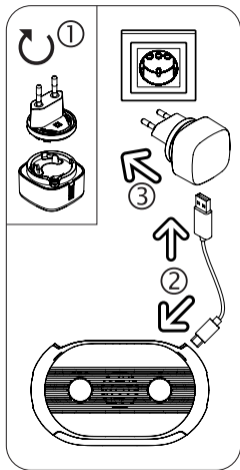

EN INSTRUCTION FOR USE

SMART mode (90, 120 or 180 minutes)

Pressed, the drying time will be automatically set between 90, 120 or 180 minutes based on the current drying space temperature and humidity measured by the integrated sensor.

MAX mode (4 hours)

In this mode the drying time is manually set to 4 hours if the hearing care professional has recommended the maximum drying time e.g. due to heavy perspiration.

Operation

1. Insert your country-specific adapter into the USB power supply by rotating it clockwise into the slot.
2. Plug the USB cable into the USB power supply unit (5V/2A) and the USB-C plug into the USB-C port (IN) on the back of your dry-cap uv 4®.
3. Then plug the power supply unit into a mains socket.
4. Place the dry-cap uv 4® over the charging station with the rechargeable hearing aids. If the hearing aid charging station has a lid, please open the lid before drying.

5. Press the desired button (SMART or MAX mode) and the care program (warm drying + UV-C) starts automatically.

① LED 1, 2, 3 or 4 light up

- The fast drying by fan with warm air (up to 38°C / 100°F) is controlled by a microprocessor and runs for 90, 120, 180 minutes in SMART mode or 4 hours in MAX mode depending on the mode.
- During drying ultraviolet LED light (275 nm) eliminates germs and bacteria to 99.9% at the beginning and before the end of the drying time.
- After drying and hygiene the dry-cap uv 4® automatically shutdown to stand-by. ① LED 1, 2, 3 or 4 switch off

Drying and hygiene are complete, the charging of the hearing aids in the charging station continues and can remain under the dry-cap uv 4® after the charging station is switched off.

Optional charging feature

The dry-cap uv 4[®] has a second USB-C port for output (OUT) at the backside. This can be used for charging the hearing aid charger. Please note, the output power is limited to 500mA. LED 5 will light up green when the power is less or equal to 500mA and LED5 will turn-off if the connected device need more than 500mA power. In this case, please use a separate power supply.

Maintenance and operation

We recommend wet wipes for cleaning your dry-cap uv 4[®]. Please also read the care and user instructions for your hearing aids and charging station. The dry-cap uv 4[®] is very energy efficient; unplug the power supply only for prolonged absence. You can interrupt the drying process in both modes by holding the button for more than 3 seconds or by lifting and tilting the unit.

! Warnings

Avoid contact with water. Don't use the dry-cap uv 4[®] near basins etc. In the event of a malfunction, unplug the power supply and take the dry-cap uv 4[®] to your retailer. Don't disassemble the dry-cap uv 4[®] and use only the supplied 5V power adapter. Children must be supervised to ensure that they don't use the dry-cap uv 4[®] as a toy. The dry-cap uv 4[®] is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the dry-cap uv 4[®], unless they are supervised for their own safety by a responsible person or receive instructions on how to use the dry-cap uv 4[®].

CAUTION: Do not look into the UV-C light, as this can harm your eyes and your skin. The dry-cap uv 4[®] has a 3G-safety sensor. If the dry-cap uv 4[®] is moved, rotated or tilted during use, the 3G-Senor switches the device off! In idle mode, you must then restart the device. Please use only with UV-C cleanable products, never with humans and animals!

Disposal

Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and wellbeing.

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© 2010-2024 flow-med GmbH. All rights reserved.
flow-med GmbH
Otoplastic AG