**GEBRUIKSAANWIJZING
MODE D'EMPLOI
GEBRAUCHSANWEISUNG
USER MANUAL
BRUKSANVISNING
KÄYTTÖOHJE**

Inhoud

Nederlands	3
Français	9
Deutsch	15
English	21
Norsk	27
Svenska	35
Suomi	41

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PRECAUTIONS BEFORE USE

KEEP THESE INSTRUCTIONS IN MIND:

1. Do not cover or block any ventilation openings. When placing the device on a shelf, leave 5 cm (2") free space around the whole device.
2. Install in accordance with the supplied user manual.
3. Keep the device away from heat sources such as radiators, heaters, stoves, candles and other heat-generating products or naked flame. The device can only be used in moderate climates. Extremely cold or warm environments should be avoided. Working temperature between 0° and 35°C.
4. Avoid using the device near strong magnetic fields.
5. Electrostatic discharge can disturb normal usage of this device. If so, simply reset and restart the device following the instruction manual. During file transmission, please handle with care and operate in a static-free environment.
6. Warning! Never insert an object into the product through the vents or openings. High voltage flows through the product and inserting an object can cause electric shock and/or short circuit internal parts. For the same reason, do not spill water or liquid on the product.
7. Do not use in wet or moist areas such as bathrooms, steamy kitchens or near swimming pools.
8. The device shall not be exposed to dripping or splashing and make sure that no objects filled with liquids, such as vases, are placed on or near the apparatus.
9. Do not use this device when condensation may occur. When the unit is used in a warm wet room with damp, water droplets or condensation may occur inside the unit and the unit may not function properly; let the unit stand in power OFF for 1 or 2 hours before turning on the power: the unit should be dry before getting any power.
10. Although this device is manufactured with the utmost care and checked several times before leaving the factory, it is still possible that problems may occur, as with all electrical appliances. If you notice smoke, an excessive build-up of heat or any other unexpected phenomena, you should disconnect the plug from the main power socket immediately.
11. This device must operate on a power source as specified on the specification label. If you are not sure of the type of power supply used in your home, consult your dealer or local power company.
12. Keep away from animals. Some animals enjoy biting on power cords.
13. To clean the device, use a soft dry cloth. Do not use solvents or petrol based fluids. To remove severe stains, you may use a damp cloth with dilute detergent.
14. The supplier is not responsible for damage or lost data caused by malfunction, misuse, modification of the device or battery replacement.
15. This device is not intended for use by people (including children) with physical, sensory or mental disabilities, or a lack of experience and knowledge, unless they're under supervision or have received instructions about the correct use of the device by the person who is responsible for their safety.
16. This product is intended for non-professional use only and not for commercial or industrial use.
17. Make sure the unit is adjusted to a stable position. Damage caused by using this product in an unstable position vibrations or shocks or by failure to follow any other warning or precaution contained within this user manual will not be covered by warranty.
18. Never remove the casing of this device.
19. Never place this device on other electrical equipment.

20. Do not allow children access to plastic bags.
21. Only use attachments/accessories specified by the manufacturer.
22. Refer all servicing to qualified service personnel. Servicing is required when the device has been damaged in any way, such as the power supply cord or the plug, when liquid has been spilled or objects have fallen into the device, when the device has been exposed to rain or moisture, does not operate normally, or has been dropped.
23. Never directly fill the reservoir with flammable or aggressive chemicals (acids, etc.), but use a separate glass container (not included).
24. Never reach into the cleaning liquid during the ultrasonic cleaning process.
25. Never use the device for cleaning living beings or body parts (fingernails).
26. If you've noticed any transportation damage, do not connect the device to the mains.
27. Insert heavily polluted objects with the dirty side downwards and don't stack them. You may need to repeat the cleaning process.



PRINCIPLE OF ULTRASONIC CLEANING:

1. It produces millions of tiny air bubbles per second, which are blasted by the high frequency mechanical vibrations.
2. Generating strong impact and negative pressure suction, so that stubborn dirt split into small granular and off the surface of the object into the water.


PRINCIPLE OF UV-C LIGHT STERELISATION

1. When the organic pollutants pass through the ultraviolet lamp irradiation area, the ultraviolet light penetrates the cell membrane and nucleus of the organisms.
2. The organism will absorb the ultraviolet light, through a fotochemical process the DNA / RNA of the organism will be changed.
3. Through the change of genetic material the activity of the organism will be removed. The organism can not reproduce and will die soon.

CONTENT: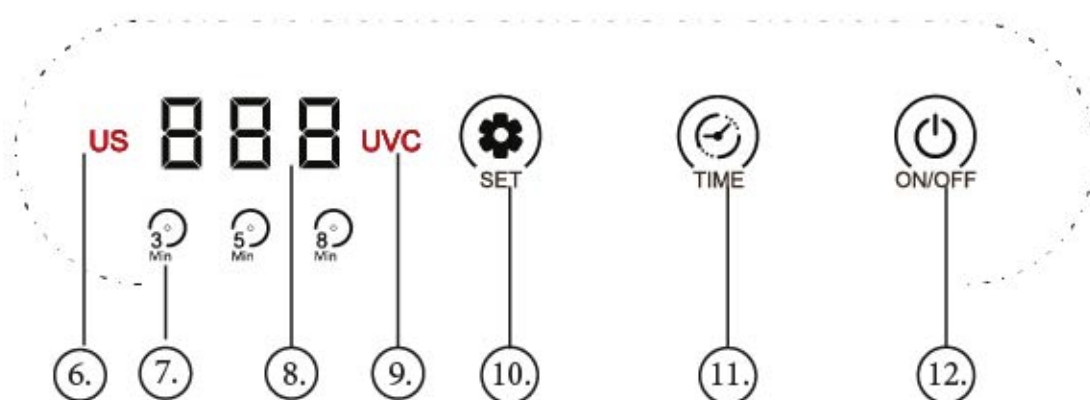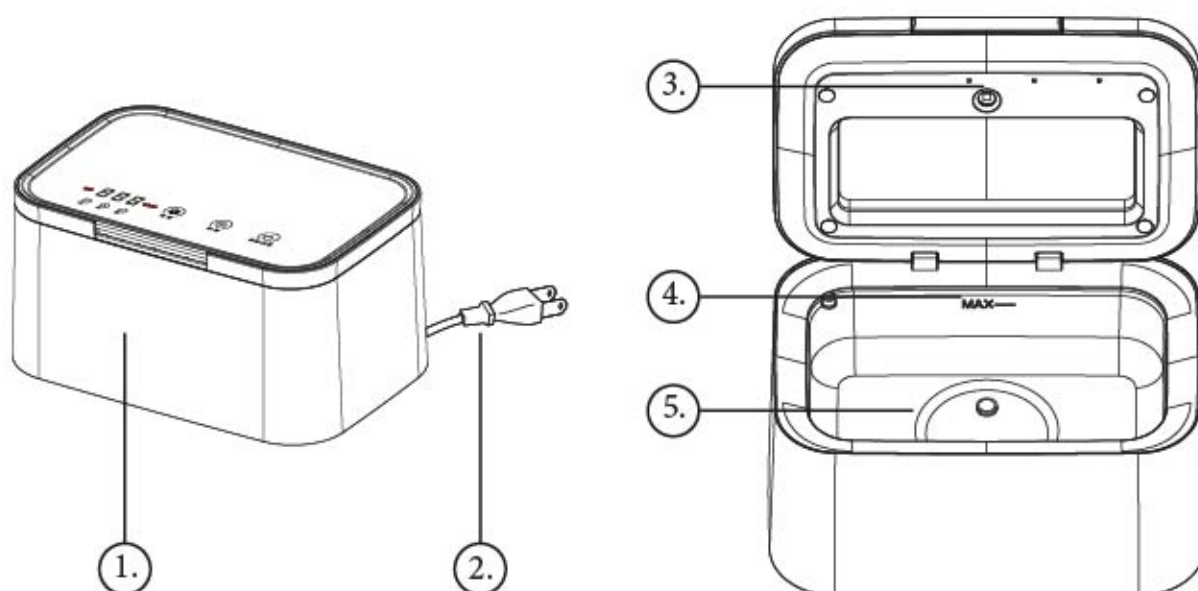


13.

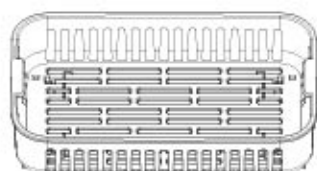14.


15.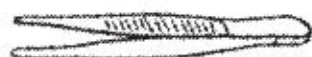

1. Base unit
2. AC powerplug
3. UV-C LED
4. Max fluid level
5. Tank
6. Ultrasonic indicator
7. Time indicator
8. Led display
9. UV-C indicator
10. Function mode
11. Time set mode
12. Start / Stop
13. Basket
14. Multi tool
15. Tweezer

PANEL CONTROL

1. The FC35 works on a 1 - 2 - 3 principle where within 3 steps the device is operational.
Select the mode through the SET button. Choose between Ultrasonic, UV-C or the combined power of Ultrasonic and UV-C. The selected mode will be visibly highlighted on the top of the lid.
2. Select the operating time. Choose between 180 - 300 - 480 seconds. The selected time is visible on the LED screen in seconds and on the Time indicator.
3. When both the function and time is selected the FC35 is ready for operation. operation can be executed by pressing the Blue On/Off button. At which point the Led display will start counting down from the selected time.
4. After operation when the device reaches 000, the FC35 will turn off automatically.
5. The manufacturer advises to let the device cool for a minimum of 5 minutes before starting a new operation.

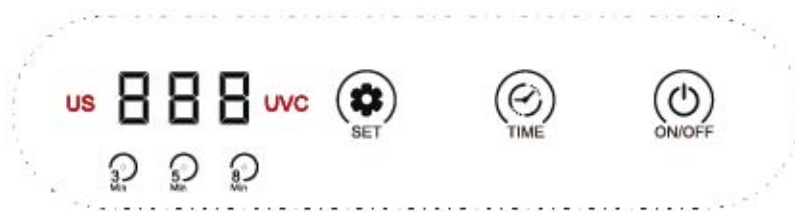OPERATING INSTRUCTIONS

1. Open the lid and fill the tank with water.
Make sure the water level does not exceed the Max fill indicator in the tank.
2. Put the to be cleaned item either in the basket or attach to the multitool.
3. Lower the basket / multi tool in the water, making sure the item is completely below the surface.
Close the lid and press prepare the FC35 to operate through the Panel control.

This product is divided into two function modes: Dry and Wet

UV-C sterilisation: Dry, Wet

When only using the UV-C function to sterilize items, there is no need to fill the tank with water. During operation, when the lid is raised the UV-C light will suspend while the timer keeps running. Close the lid for the UV-C light to turn back on and resume sterilisation operation.

Ultrasonic cleaning: Wet

When using the Ultrasonic function, always make sure that the tank is sufficiently filled to cover the product while not exceeding the MAX level. When in operation the lid can be opened and closed without disturbing the ultrasonic process.

Ultrasonic + UV-C: Wet

When using the Ultrasonic function, always make sure that the tank is sufficiently filled to cover the product while not exceeding the MAX level. During operation, when the lid is raised the UV-C light will suspend while the timer and the ultrasonic cleaning process continues. Close the lid for the UV-C light to resume.

EXAMPLES OF SUITABLE OBJECTS FOR CLEANING

1. Glasses and spectacle frames (avoid direct contact between the spectacle glasses and the bottom of the reservoir to prevent scratching. Use the basket with the glasses facing up). Pay attention to any loose pieces or any damage to the coating of the frame or to the glass. In this case, the coating may be further damaged during the cleaning process.
2. Contact lenses (hard and soft) in combination with multi-purpose no-rub liquid. Pay attention while cleaning glasses. An already loosened coating or colour layer on the glass or frame may be further loosened when using the cleaner. When in doubt, don't use the cleaner for glasses with coating.
3. Gold or silver jewellery, gemstones, emeralds and pearls.
4. Felt-tip pens, ink cartridges and accessories.
5. Dentures, brackets, instruments, etc.
6. Shaving heads of electric razors.
7. Delicate mechanical parts of e.g. clocks or for model building.

MAINTENANCE

1. Do not clean the FC35 with harsh cleaning products or sharp objects.
Keep the device in a cool, dry place, well out of reach of children.
2. Do not leave the water in the cleaning container. Empty immediately after each use and make sure the FC35 powerplug is unplugged.
3. Clean the device with a damp cloth only.
Do not submerge the FC35 in water.

DISCARDING OLD EQUIPMENT

Old electric and electronic devices often contain valuable materials. However, they also contain harmful substances that were necessary for their functionality and safety. If these are placed with the non-recyclable waste or disposed of incorrectly, they can be harmful to human health and the environment. Never place your device with the non-recyclable waste.

REMARK

- Use the collection points available in your community and recycle old electric and electronic devices.
Contact your town hall, the local waste disposal company or your dealer for more information.
- Make sure that your old device, until it's picked up, is safely stored out of reach of children.

WARRANTY

- We provide a warranty of 24 months for this product. The warranty starts on the date of purchase and is intended for faults due to production or the used materials.
- Your statutory warranty rights in accordance with § 439 ff. BGB-E will remain unaffected by this.
- The warranty does not apply to damages as a result of improper handling or use, nor to malfunctions that only have a minor effect on the functionality or value of the product. Normal wear and tear, transport damage, to the extent that we are not responsible for this, as well as damages caused by repairs not performed by us, are also excluded from the warrant.

- This device is designed for household use and has the right performance levels.
- Any use in commercial situations is only covered by the warranty if the device is not put under a heavier strain than in a household situation. The device is not intended for additional commercial use.
- In the case of well-founded complaints, we will repair the defective device at our discretion or replace it with a device that shows no problems.

All possible manufacturing faults must be reported within 14 days of delivery. All further claims are excluded. When invoking the warranty, please contact us first before sending us the product (you must always provide us with proof of purchase).

SPECIFICATIONS

- Model: FC35 - ultrasonic Cleaner + UV-C Sterilizer
- Voltage: 220V-240V; 50-60Hz
- Power consumption: 50W
- Capacity MAX line: 600ml
- Product dimensions (W*D*H): 210 * 140 * 115mm
- Weight: 950g

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Commaxx B.V., Wiebachstraat 37 6466 NG Kerkrade, The Netherlands