
SWIMSAFE

KEEP WATER OUT OF YOUR EARS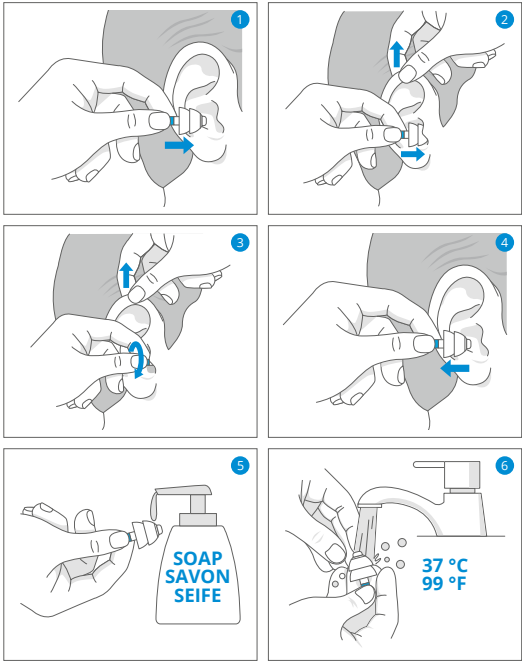

ATTENUATION DATA Alpine SwimSafe
SwimSafe is no hearing protection according to the regulation EU 2016/425

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Certified and monitored by PZT GmbH, Bismarckstr. 264B, 26389 Wilhelmshaven, 91974

EN The hearing protector is intended to protect the wearer from water entering the ears. The wearer should ensure that the earplugs are fitted, adjusted and maintained in accordance with this Instruction Manual. Any other uses are not intended and therefore not allowed. Slightly pull up your ear and with your other hand insert the earplug into the ear, until the protector fits snugly. Do not place the earplug too deep in the ear. Do not use beyond a depth of 1 metre underwater. Rinse thoroughly and dry off after use. Improper fit, improper use or failure to follow the instructions of this device will affect the lifespan of the reusable earplugs. Alpine Hearing Protection is not liable for any damage arising from misuse. Check the working order of the earplugs regularly and store them in the original, clean packaging. Clean the earplugs regularly with mild soap and lukewarm water. Dry off after cleaning. The reusable earplugs can be used approximately 100 times. Sudden or rapid removal of hearing protection may cause damage to the eardrum. Do not remove plugs from the ear by pulling the connecting cord. Do not use a connecting cord if there is a chance of getting entangled. This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer. Hereby, Alpine Hearing Protection, declares that the hearing protector SwimSafe is in compliance with the PPE regulation (EU) 2016/425. The full text of the EU Declaration of Conformity is available at the following internet address: www.alpine.eu/doc

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